




Artificial Insemination & Embryo Transfer for Cow

Intra-uterine horn catheter

“mo-No.5”

The Legend “Shimura System”

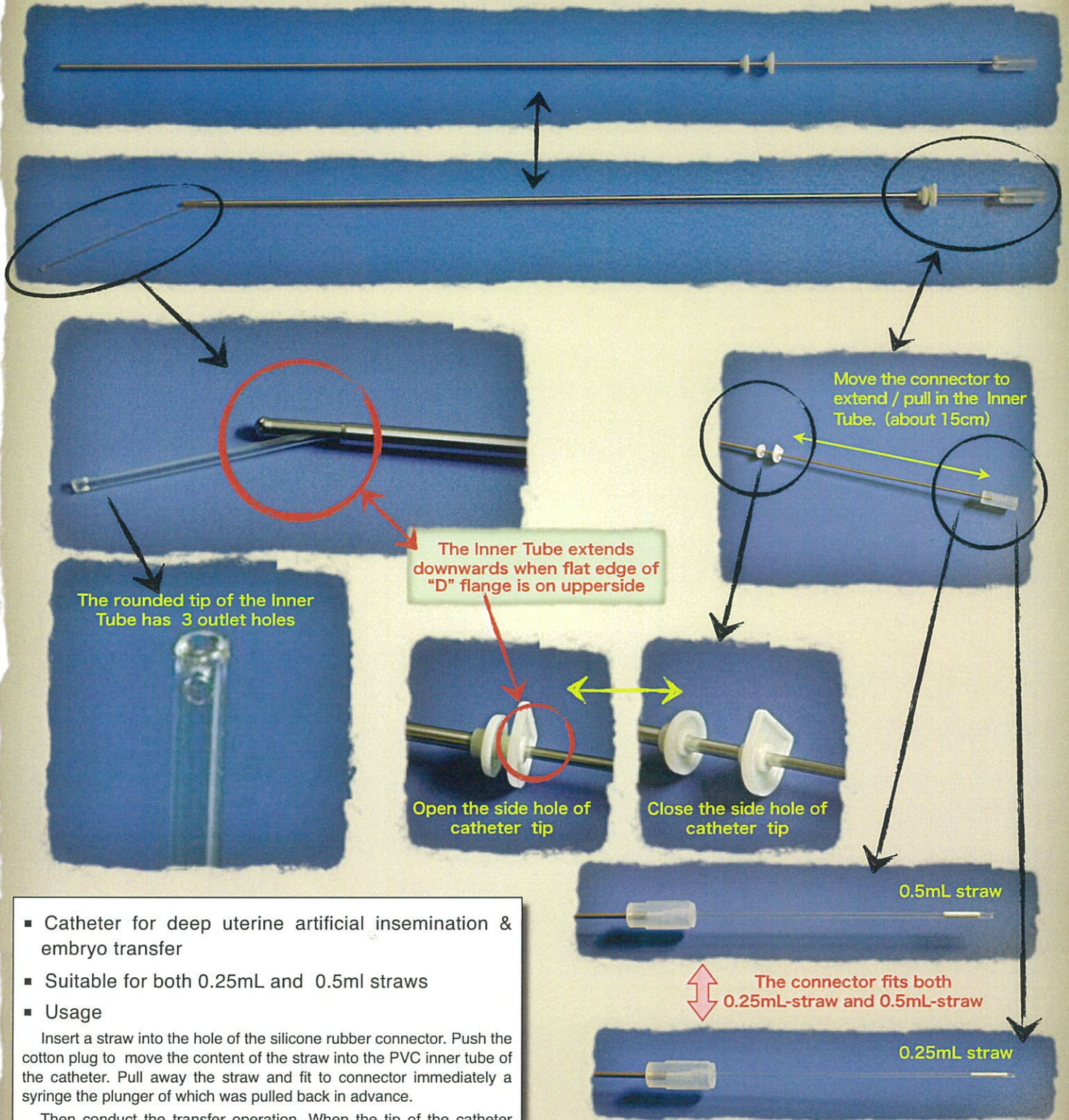

- Catheter for deep uterine artificial insemination & embryo transfer
- Suitable for both 0.25mL and 0.5ml straws
- Usage

Insert a straw into the hole of the silicone rubber connector. Push the cotton plug to move the content of the straw into the PVC inner tube of the catheter. Pull away the straw and fit to connector immediately a syringe the plunger of which was pulled back in advance.

Then conduct the transfer operation. When the tip of the catheter reaches the uterine corn, push the "D" flange to open the side hole of the catheter tip, and further push the connector to extend the inner tube. Then push the plunger of the syringe to release the content in the inner tube.

- Packing: Individually packed
- Packaging: 20 sets / box

MISAWA MEDICAL INDUSTRY CO., LTD.

351, Asahi-machi Kasama-shi, Ibaraki-ken, 309-1717 Japan

Tel : +81-296-77-8804 Fax : +81-296-77-8849